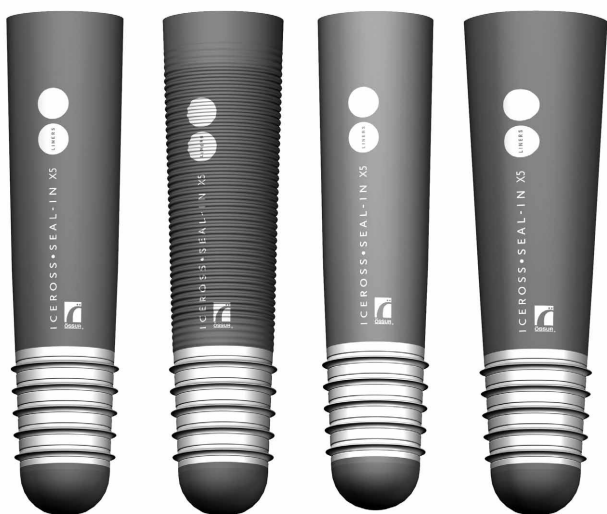
Instructions for use

Iceross Seal-In® X5

For TT/TF users

Life Without Limitations®

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	9
ⓕⓇ Guide de fabrication.....	13
ⓔⓈ Instrucciones para el uso.....	18
ⓔⓣ Istruzioni per l'uso.....	22
ⓓⓐ Brugsanvisning.....	27
ⓈⓋ Bruksanvisning.....	31
ⓕⓖ Käyttöohjeet.....	35
ⓔⓁ Οδηγίες χρήσης.....	39
ⓃⓁ Gebruiksaanwijzing.....	44
ⓔⓣ Instruções de Utilização.....	49
ⓙⓐ 取扱説明書.....	54
Ⓩⓗ 中文说明书.....	58
Ⓚⓞ 사용 설명서.....	62

EN - Caution: Ossur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Ossur products and components are used with other recommended Ossur or authorized components. If unusual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.

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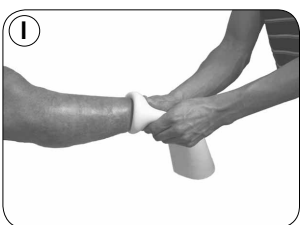
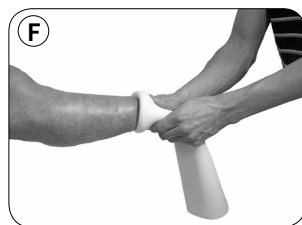
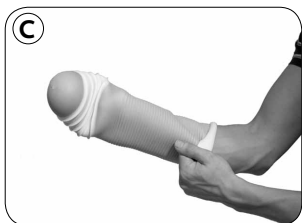
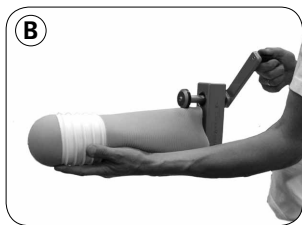
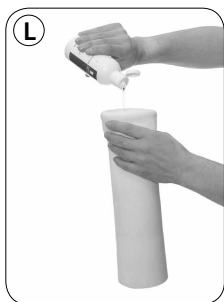
ENGLISH

A. INTRODUCTION

Thank you for choosing a high quality silicone liner from Ossur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

B. CONTRAINDICATIONS FOR USE

Contraindications are:

1. Unstable limb volume.
2. Extremely conical limbs (see Section J).
3. Very short residual limbs.

Minimum recommended length:

Transtibial: To assess the minimum recommended length, measure the residual limb length from the mid patella tendon to the distal end, with knee in full extension.

Transfemoral: Measure the residual limb length with the tissue hanging down, from the perineum to the distal end.

- Compare the measurement to the table below.
- Low trimlines of the hard socket can affect suspension and should be assessed on individual basis.

Liner Sizes	Minimum Length
18 - 23	11,0 cm
25 - 30	12,5 cm
32 - 40	13,5 cm
45 - 50	15,5 cm
55	16,5 cm

NOTE: Low posterior trim lines of the hard socket can affect suspension and should be assessed on individual basis.

C. LINER SIZING: (Figure A)

To determine the correct Iceross size:

1. Measure the circumference of the residual limb at 4cm from the distal end with the tissue hanging down.
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. **DO NOT** choose a liner size, which exceeds the measurement, or round up.

EXAMPLE: If the residual limb measures 31cm at the indicated location, choose size 30 **NOT** size 32.

Selecting the correct size is very important.

If the liner is too tight, this may cause pistoning and/or numbness and blistering. A Seal-In liner that is too tight will also produce excessive ring tension to the residual limb from the seals. If the liner is too loose, increased perspiration and/or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be instructed to contact his/her physician or prosthetist immediately.

D. TRIMMING AN ICEROSS LINER: (Figure B)

Iceross liners may be trimmed for user preference and to increase range of motion. The Iceross CurveMaster is specifically designed for this func-

tion and to produce a smooth, rounded edge, which will minimize the risk of skin irritation and/or tears of the liner.

Do not cut the liner below or through any of the Seals or below the proximal trim-lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and/or compromise suspension.

E. DONNING THE ICEROSS SEAL-IN® X5

To don the liner use the Clean & Simple Lubricant Spray (see Section H). Before donning a new liner clean it with lukewarm water, and dry it with lint free cloth as the natural oils in the silicone make it difficult to grip.

1. Turn the liner inside out by gripping the distal end from the inside as shown in the illustration. **(Figure C)**
2. Grip the top of the liner and slide it over the hand until the inner surface is fully exposed. **(Figure D)**
3. Spray lubricant into the inverted liner for easier roll on. **(Figure E)**
4. After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present. **(Figure F)**
5. Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull.
6. Ensure the seals sit horizontally around their full circumference.

NOTE: Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

NOTE: With use the silicone will soften, which will facilitate application.

F. DONNING THE PROSTHESIS

The Iceross Seal-In X5 liner is to be used with Icelock Expulsion Valves 551 or 552 (see Section I). The valves are specifically designed to maintain suspension when using Seal-In liners. A knee sleeve is not required.

1. Lightly spray the Seals around their full circumference, and inside the hard socket for smoother entry. **(Figure G)**
2. Align the prosthesis as instructed by the prosthetist and push into the hard socket expelling air. **(Figure H)**

NOTE: If the user is having difficulty donning the prosthesis, more lubricant may be needed or the valve is blocked.

NOTE: If the user feels the skin of the residual limb is being stretched to one side after donning the prosthesis, the distribution of the lubricant is inadequate around the full circumference of the Seals.

G. VOLUME SOCKS

If volume reduction is experienced in the residual limb, volume socks may be added to restore socket fit.

Volume sock application:

- Pull the volume sock over the Seals and ensure it is placed level between the two most proximal Seals.
- The volume sock can extend over the top two Seals as long as suspension is maintained.
- Don the prosthesis as normal.

NOTE: Placing volume socks over the three distal Seals will result in loss of suspension.

H. ICEROSS® CLEAN & SIMPLE LUBRICANT SPRAY

Iceross Clean & Simple lubricant spray has to be used to facilitate donning of the liner and the prosthesis.

The lubricant is designed to evaporate quickly after donning to maintain maximum suspension. A spray pen accompanies the Seal-In Liner, small enough to fit in a pocket and/or a handbag. It can be refilled with

the Iceross Clean & Simple lubricant spray.
Warning: Do not apply to the skin. Avoid contact with eyes.
Ingredients: Propanol and water.

I. ICELOCK® EXPULSIONS VALVES

The Ossur Icelock Expulsion Valves are designed and tested to continually expel air when loading the prosthesis, thus maintaining hypobaric suspension.

PRODUCT NUMBER	DESCRIPTION	BENEFITS
L-551000	Expulsion Valve 551	Small, cosmetic & easy installation
L-552000	Expulsion Valve 552	Removable valve and large smooth opening

J. ICEROSS® DISTAL CUP

Iceross Distal Cups (I-3000xx) provide enhanced comfort for conical limbs with poor soft tissue coverage. They can also be used as a fitting tool in conjunction with the Iceross Original liners to ensure total contact between the silicone and residual limb. The DERMOGEL silicone in the Iceross Distal Cup conforms to the limb, thus minimizing pressure over bony prominence during gait.

To correctly fit a Distal Cup:

1. Measure the circumference of the residual limb at 4cm from the distal end. **(Figure A)**
2. Choose the Distal Cup two or three sizes smaller than the measurement
3. Roll the Distal Cup directly over the limb. **(Figure I)**
4. When using an Iceross liner in combination with a Distal Cup, the liner measurement (4cm from the distal end) should be made with the Distal Cup in place (i.e. OVER the Distal Cup). **(Figure J)**
5. Choose the correct Iceross liner size based on the measurement obtained with the Distal Cup in place.

K. ICEROSS® PADS: (Figure K)

Iceross Pads (I-6103xx) can be used to fill in any problem areas (i.e. voids or scars) to ensure better fit and to minimize trapped air. Iceross Pads may also be used to distribute body pressure and to protect the skin. In some cases, custom-formed pads may be required for irregular shapes or problem areas. See Ossur Prosthetic Product Catalogue for more information.

L. LINER CARE: (Figure L)

Proper product care is important. The liner needs to be washed inside and out every day, following use. Your patient should be instructed to remove the liner, turn the liner inside out and wash it with Iceross Clean & Simple soap **(See Section M)**. The liner can also be machine washed (40°C/Hot) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should **NOT** be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry over night. Always return the liner to its normal position with the distal attachment facing out as soon as possible after cleaning. The Seals must be clean and free from dirt prior to use.

Iceross liners should always be checked for damage or wear. Any damage may weaken the effectiveness of the liner and should be reported to the user's prosthetist immediately. Misuse may result in potential loss of suspension.

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Iceross Seal-In® X5 is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.
Össur Iberia S.A.